



# MediZysis India Pvt. Ltd.

one stop solution

# Rustguard™

WE WORK 24X7, 365 BECAUSE RUST NEVER SLEEPS



## Composition

DDAC+ BKC+ORGANIC ACIDS+  
INORGANIC ACIDS+ NON-IONIC  
SURFACTANTS

**RUSTGUARD** is a combination of highly effective chemicals which breaks rust chemically and allows it to dilute and disintegrate from the instruments. Its other ingredients like DDAC makes it a strong antimicrobial agent which keeps the instruments free from Bacteria, viruses, fungi and other micro-organisms. Prolonged uses of **RUSTGUARD** makes instrument smooth and makes it free flowing.

## FEATURES:

- ♦ 100% biodegradable and Nontoxic.
- ♦ Removes Rust from instruments
- ♦ Non-carcinogenic.



**MediZysis India Pvt. Ltd.**  
**transforming hygiene ...**

B-1/209, Harsh Vihar, Delhi-110093  
Customer Care: 011-22343535, 9425628989  
Email: medizysis@gmail.com



## Directions For Use

20 ml RUSTGUARD is to be mixed with hot water (around 40-50C).Submerge the instrument completely in this solution for around 15-20 minute and then rub gently and clean properly with normal water afterwards. Do not leave any instrument in the solution of RUSTGUARD for more than 20 minutes. Freshly prepared solution is always recommended.

## Dilution for Uses

- ◆ 20 ml per litre for normal rust,
- ◆ Put directly on rust for stubborn rust.

## Material Compatibility

RUSTGUARD is compatible with most of the metals.

<b>Colour</b>	- colourless
<b>Odour</b>	- Odourless
<b>PH</b>	- 7.0 (Approx)
<b>Contact time</b>	- 15-20 minutes
<b>Shelf life</b>	- 2 years.
<b>Storage</b>	- Room Temperature

## Hospital Areas of Application

- ◆ CSSD, OT
- ◆ Surgical Units, Dental Units
- ◆ Endoscopy / GI Department
- ◆ ENT Department
- ◆ Anaesthesia Department

## Precautions

- ◆ For External Use only.
- ◆ Avoid direct contact with eyes & skin.
- ◆ Rinse with fresh water in case of contact.
- ◆ Seek medical help in case of emergency.
- ◆ Keep out of reach of children



**MediZysis India Pvt. Ltd.**  
transforming hyziene ...

B-1/209, Harsh Vihar, Delhi-110093  
Customer Care: 011-22343535, 9425628989  
Email: medizysis@gmail.com



**Name of Product - RUSTGUARD**  
**Pack Size - 1000 ML**